



Digital Dentures

What Are Digital Dentures?

Digital Dentures are dentures fabricated by CAD design using 3D printing and 5 axis milling technologies.



Digital Dentures

The Benefits

- **FAST** Reduced turnaround time. You save approximately 16-21 business days of manufacturing.
- **BETTER FIT** CAD/CAM technology is extremely accurate and eliminates human error to achieve a better fit.
- **DIGITAL BACK-UP** If dentures are damaged or lost, new ones can be remade at any point.

How does it work?

Option # 1

3 Appointments for Edentulous Patient

Option # 2

2 Appointments for Patient with Existing Dentures

Option # 1

3 Appointments for Edentulous Patient

APPOINTMENT # 1 - INITIAL SCAN

Lab will return digital wax rims
within 5 business days.

Option # 1

3 Appointments for Edentulous Patient

APPOINTMENT #2 - BITE REGISTRATION

Using the bite blocks, take a bite registration. Mark the lip line, midline, and canine position. If necessary (if fit is not retentive enough), relin the base with medium body impression material. Lab will return a finished denture within 6-7 business days

Option # 1

3 Appointments for Edentulous Patient

APPOINTMENT #3 - DELIVERY

Delivery of completed denture.

If any fit or bite adjustments are required that cannot be done chairside, please take a new bite using the dentures and a medium body wash impression, and send back to the lab to be reprinted.

Approximately 12 business days of in lab production.

Option #2

2 Appointments for Patient with Existing Dentures

APPOINTMENT # 1

- 1) Reline existing dentures in patient's mouth
- 2) Take a bite with existing dentures
- 3) Using oral scanner, scan the dentures and the bite. Then send the scan to the lab.

Option #2


2 Appointments for Patient with Existing Dentures

APPOINTMENT #2

Final Denture Delivery

Approximately 7 business days of in lab production.

The Conventional Process

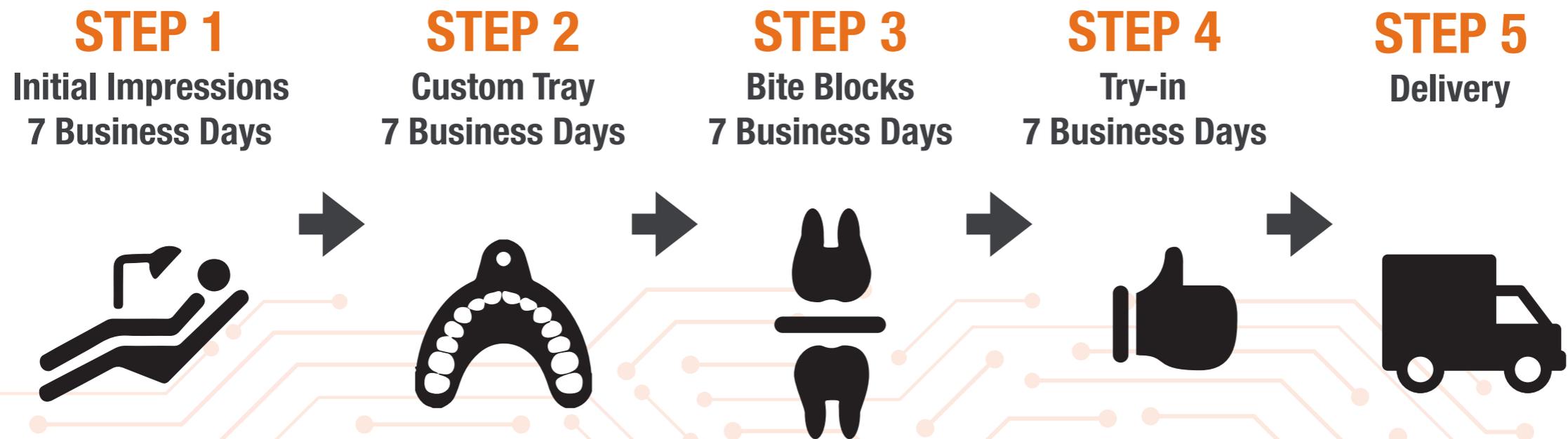


The conventional process of digital dentures will take 28 in lab business days (not including shipping) from beginning to end.

The Digital Denture Process

STEP 1
Initial Scan
5 Business Days



STEP 2
Custom Tray/Bite Blocks
as one piece
6-7 Business Days

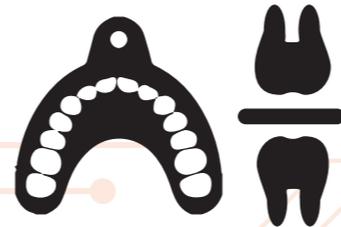

STEP 3
Final Delivery



The express process is shortened by skipping an appointment from the conventional process. Doctor can request teeth try-in between appointment #2 and #3*.

*Please note that this will incur an extra charge to cover the cost of printing a teeth try-in



Call 224-754-3100
Email info@gcdl.net
Visit www.GCDL.net

